6 Quick Steps to Recharge Your Battery

INOGENONE®G3
External Battery Charger

Your Equipment Provider:

Your Physician:



326 Bollay Drive, Goleta, CA 93117 1-877-4-INOGEN (Toll Free) 1-877-446-6436 (Toll Free) 1-805-562-0515 (International) info@inogen.net www.inogenone.com www.OxygeNation.com







Plug the External Battery Charger AC power supply cord into the nearest electrical outlet.

INOGENONE®G3 External Battery Charger

Your Inogen One G₃ External Battery Charger (BA-303) will charge your Inogen One G₃ single and double batteries. It takes approximately 3 hours to charge a fully depleted single battery, and approximately 6 hours to charge a fully depleted double battery. Note: the battery will also recharge in the Inogen One G₃ when powered by the AC Power Supply, or DC Power Cable.



Plug the External Battery Charger AC power supply into the battery charger.





Slide the charger onto the Inogen One G₃ Battery by clicking and locking into the charger.



When the battery is in the correct position, a solid red light will indicate that the battery is charging.



Press battery latch down and slide charger off battery.



When the green light illuminates, the battery is fully charged*.



*If the red light is flashing, unplug the unit and complete steps 2-3 again. If a red flashing light continues after these steps, contact your Equipment Provider.